



## ABOUT US

American Surgical Company is a global medical technology company focused on developing and selling creative and smart surgical technologies. ASC recognizes the challenges of surgical procedures, and meets those challenges by working directly with physicians, operating room personnel, and building corporate partnerships. ASC has established a global presence across six continents.

THE COMFORTPACK IS PATENT-PENDING.  
FEDERAL (USA) LAW RESTRICTS THIS DEVICE TO SALE  
BY OR ON THE ORDER OF A PHYSICIAN OR THEIR  
REPRESENTATIVE. FOR AVAILABILITY OUTSIDE OF  
THE UNITED STATES, PLEASE CONTACT US.

MM4161.003

*"The ComfortPack is fast and effective. The ComfortPack doesn't only eliminate pain it eliminates the feeling of pressure, making it the best choice. 95% of our patients find the ComfortPack much more comfortable than what we were previously doing."*

**Diana Lozowski RN, Claris Vision**

*"Once I began using the ComfortPack, my patients began to consistently tell me that my injections were some of the most comfortable they experienced. ComfortPack works above and beyond any other approach."*

**Dr. Michael McAllister M.D., Confluence Health**

# ComfortPack™

THE PATIENT PREFERRED CHOICE

[www.ComfortPack.co](http://www.ComfortPack.co)

1-800-343-0060

45 Congress St. Salem, MA 01970

# ComfortPack™

Needle-free Ocular Anesthesia



THE PATIENT PREFERRED CHOICE

## HOW IT WORKS


### *Dip pledgets in 4% lidocaine*

Pour enough lidocaine in the cup to saturate the pledgets. Typically, around 0.5 – 1 ML.



### *Put pledget in patient's fornix*

Using the applicator, remove the pledget from the cup and place it in the desired injection area.

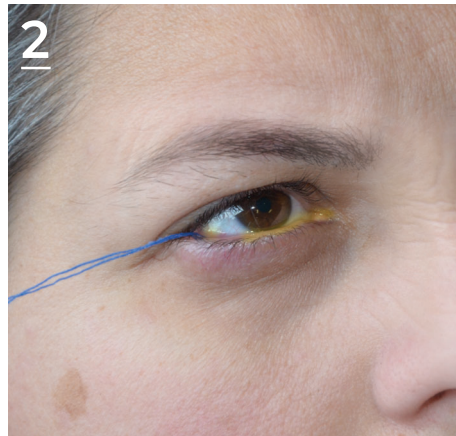

### *Remove pledget from eye in minutes\**

Remove the pledget by pulling on the string and begin the injection process.



*\* Typically 5-7 min. Physician discretion.*

*PLEASE CONSULT THE PACKAGE INSERT  
FOR MORE INFORMATION.*

**For more instructions please visit:  
[www.comfortpack.co/training](http://www.comfortpack.co/training)**

## WHY IT'S BETTER

- ✓ **Substantially increases patient comfort**
- ✓ **Reduces needle anxiety and fear**
- ✓ **Lowers the risk of infection and subconjunctival hemorrhage**
- ✓ **Can be applied by nurses & ophthalmic assistants**
- ✓ **Saves time and money**
- ✓ **Easy to use, fast & efficient**

*For questions or to place an order*

**Call: 1-800-343-0060**

**Visit us online at: [www.comfortpack.co](http://www.comfortpack.co)**